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06022020 Performance indicator is changed

1. Goal

To describe how heparin plasma (4 ml) must be prepared and stored for the purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol heparin plasma 4 ml

Collection

Fasting patient Dependent on the requirements of each Pearl/sub-biobank.

Delivery In collection tube.

Type of tube 4 ml lithium heparin tube, without gel, without protease inhibitors.
BD tube 367883 or equivalent.

Temperature Keep the tubes at room temperature until preparation.

Preparation

Time until freezing Prepare and freeze the collected blood as soon as possible.
Aim: within 2 hours.
Maximum: within 4 hours.

Centrifuge g-force: 2000g
Time: 10 minutes.
Temperature: At room temperature.

Storage

Register Register the microtubes in the biobank management system and attach a cryovial label.

Aliquotation Store the biomaterial in 0.5 ml aliquots to a maximum of 3 aliquots. If there is biomaterial left over add it to the 3 aliquots.

Type of tube 2 ml polypropylene microtube and screw-top with O-ring.
GREI722301UMC and GREI368380UMC or equivalent.

Temperature - 80°C

Miscellaneous

Deviations Note all deviations from this procedure in the biobank management system. Assess the samples macroscopically for the following deviations (Haemolytic, Icteric and Lipemic) and register them in the Biobank management system.

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the sub Biobank samples have been prepared and stored within the parameters of this procedure.

4. Comments

-

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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