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## Revision changes

Biobank tracking system is changed in Biobank management system

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## 1. Goal

To describe how PAXgene tubes must be prepared and stored for purposes of the Radboud Biobank, so that:

1. Interested parties including potential users know how the biomaterial has been handled.
2. The laboratory can assess whether they can process the biomaterial as described under paragraph 2 "Preparation".
3. The laboratory knows the requirements for the collection, processing and storage of the biomaterial to enable registration of deviations.
4. The sub biobank knows the requirements for the collection and transport of the biomaterial to ensure correct delivery.
5. The RB is able to couple this procedure to the biomaterial in storage to determine the fitness for purpose.

### 1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank, the Sub biobank/Project and the Radboud Laboratory for diagnostics.

## 2. Protocol PAXgene tubes

### Collection

Delivery In collection tube.

Type of collection tube PAXgene tube  
BD-tube 2,5 ml 762165

Preanalysis Always collect PAXgene tube as last. If it is the only tube required use any tube to fill the interior volume of the collection set and dispose of this later. After collection invert the PAXgene tube 10 times.

Temperature Keep the tubes at room temperature until freezing.

### Preparation

Time until freezing On arrival stand at room temperature. After a minimum of 2 hours and a maximum of 72 hours at room temperature place the tube in the freezer at -20°C.

After a minimum of 24 hours and a maximum of 7 days at -20°C, store the tube at -80°C.

### Storage

Register Register the PAXgene tube in the biobank management system and attach a cryovial label. Remove first all labels from the PAX gene tube before adding cryovial label.

Aliquotation Do not divide into aliquots but freeze the complete PAXgene tube in an upright position.

Temperature -80°C

**Miscellaneous**

Comment Isolate (mi)RNA within a maximum of 11 years. After defrosting the contents of the PAXgene tube for RNA isolation, first let it stand at room temperature for 2 hours.

Deviations Note all deviations from this procedure in the Biobank management system.

### 3. Quality assurance

Review takes place via i) an internal audit (see the [audit year plan](#) in DMS) and ii) 2-monthly performance reviews of deviations registered in the biobank management system (DMS map "[Registraties en controles](#)").

#### 3.1 Performance indicator

95% of the samples of the sub biobank have been prepared within the parameters of this SOP.

### 4. Comments

-

### 5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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