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Revision changes

20-02-2020 performance indicator in changed
25-03-2019 By goal “within the framework of the Parelsnoer” is removed.
25-03-2019 Collection volume and amount of aliquots is changed
25-03-2019 Comment is added
Translation oversight corrected.

1. Goal

To describe how CSF must be prepared and stored for the purposes of the Radboud Biobank, so that:

1. Potential users know how the biomaterial has been handled.
2. The laboratory preparing the biomaterial for storage knows the minimal requirements for the handling of it.

1.1 Scope of application

This procedure is applicable to all employees concerned at the Radboud Biobank and the Radboud Laboratory for Diagnostics.

2. Protocol CSF collected in CSF tube (polypropylene) 10 ml

Collection

Delivery	In CSF tube.
Type of tube	CSF tube (polypropylene) 10 ml.
Volume	Minimum of 0,5ml, Maximum of 10 ml.
Temperature	Keep tubes at room temperature until prepared for storage.

Preparation

Time until freezing	Maximum 2 hours.
Centrifuge	g-force: 2000g. Time: 10 minutes. Temperature: Room temperature.

Storage

Registration	Register the microtubes in the biobank management system and attach a cryovial label.
Aliquotation	Store the biomaterial in 0.5 ml aliquots to a maximum of 10 aliquots. Divide left-over material when >5ml in a maximum of 5 aliquots of 1 ml.
Type of tube	2 ml polypropylene microtube and screw-top with O-ring. GREI722301UMC and GREI368380UMC or equivalent.
Temperature	-80°C

Miscellaneous

Deviations	Note all deviations from this procedure in the biobank management system.
------------	---

3. Quality assurance

Review takes place via an internal audit (see the audit plan).

3.1 Performance indicator

95% of the sub Biobank samples have been prepared and stored within the parameters of this procedure.

4. Comments

The Parelsnoer Institute SOP was based on the wishes of the NDZ Parel. The Radboud Biobank chooses to deviate from this SOP because it does not suit our wishes. See substantiation SOP liquor polypropylene buis,

5. Relevant documents

Qdocs, literature, legislation and codes of conduct, website etc.

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